beurer


FB 25

- Ⓓ **Fuss-Sprudelbad**
Gebrauchsanleitung
- ⒼⒷ **Foot Bubble Spa**
Operating Instructions
- Ⓕ **Bain de pieds bouillonnant**
Mode d'emploi
- Ⓔ **Baño de pies con masaje de burbujas**
Modo de empleo
- Ⓘ **Idromassaggio**
Istruzioni per l'uso
- ⒹⓇ **Fışkırtmalı ayak banyosu**
Kullanma Talimatı
- ⓇⓊ **Гидромассажная ванна для ног**
Инструкция по применению
- Ⓟ **Gorący masaż dla stóp**
Instrukcja obsługi
- Ⓝ **Pedilúvio de hidromassagem**
Gebruiksaanwijzing
- Ⓟ **Pedilúvio de hidromassagem**
Instruções de utilização
- ⒼⓇ **Ποδόλουτρο με αναβρασμό**
οδηγίες χρήσεως

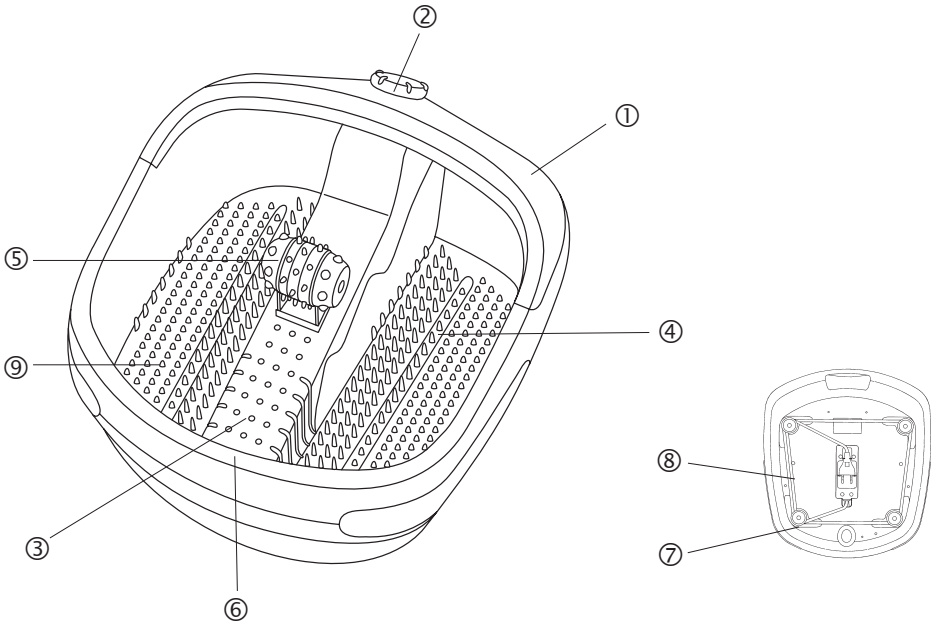BEURER GmbH + Co. KG • Söflinger Str. 218 • 89077 Ulm (Germany)
Tel.: +49 (0) 731 / 39 89-144 • Fax: +49 (0) 731 / 39 89-255
www.beurer.de • Mail: kd@beurer.de

D Deutsch

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ENGLISH

Beneficial for your feet

Our feet play a weight-bearing role in our life – they carry our entire body weight. That is why your feet want to be looked after. There are various possible methods:

- Magnetic field therapy: the positive features of magnetic fields on the human body have been known for a long time. They are used in magnetic field therapy to increase the supply of energy to the cells.
- Kneipp foot baths: alternating hot and cold baths are the foundation of Sebastian Kneipp's health philosophy.
- Foot reflex zone massage: foot reflex zone massage is a very popular massage technique, in which the entire body is believed to be influenced by treating the feet.
- Foot bubble bath: the foot bubble bath is a simple and popular method for looking after your feet at home. Using a foot bubble bath, you yourself can actively do something good for your feet as often as you like.

Foot bubble bath FB 25

This foot bubble bath is extensively equipped:

- 16 magnets integrated in the foot bath
- Massage sole
- Detachable massage roller for foot reflex zone massage
- 3 functions: vibration massage, bubble massage and water heating

Relax yourself and your feet when using this foot bath. It can have the following beneficial effects:

- stimulates the circulation
- promotes cell regeneration
- revitalises and invigorates the feet
- relaxes tense muscles

Beurer tip: warm foot baths with arnica, rosemary or sea salt extracts get the circulation going. Lukewarm foot baths with lavender or thyme have a relaxing effect and are a blessing for exhausted feet.

Instructions for use

Fill the foot bubble bath with warm or cold water up to the fill line (approx. 3 – 4 cm). Ensure that the rotary switch is set to "0" before the unit is connected to the mains.

Make yourself comfortable in a chair and place your feet on the foot rest and apply slight pressure. Never stand up in the foot bath. The rubber feet on the base of the unit prevent it from sliding.

Set the desired function using the rotary switch.

- Off (0)
- Heat and Vibration massage (W + M)
- Heat (W)
- Heat with Bubble and vibration massage (A + W + M)

A heating element is fitted below the foot surface so that hot water can be kept hot for some time. However, this heating element is not suitable for heating cold water.

To obtain an optimal effect a massage time of up to 30 minutes is recommended.

After your foot massage, turn the switch to "0" and remove the plug from the mains socket. The drain holes allow the water to be removed.

Cleaning

After use rinse the unit with tap water. The foot bath can be cleaned with a mild, non-foaming household cleaner, e.g. neutral vinegar. To dry the air lines, switch to the bubble setting without water for approx. 1 minute.

Exchangeable Massage Attachment for Massage of the Foot Reflex Zones

Additional comfort is provided by the local massage of the reflex zones. Massage of specific areas on the soles of the feet can exert an effect on organs of the body.

It is recommended that you read the specialised literature and consult a doctor before the reflex zone massage to ensure that you obtain the desired effect.

Important Information – Keep for Later Use

- **Read these instructions carefully and show them to other users.**
- Plug the unit only into the mains voltage specified on the rating plate.
- You must never connect the foot bath to the power supply when your feet are already in the water. If the device is damaged, this can lead to life-threatening electric shock.
- Ensure that the foot spa is undamaged before using it. If any fault or malfunction should occur during use, switch the unit off immediately and remove the plug from the mains socket. Never attempt to repair the unit yourself. Improper repairs can expose the user to considerable risks.
- If the mains lead of the unit is damaged this must be replaced by a service centre as special tools are required.
- The unit is intended only for the purpose described in the instructions for use. The manufacturer cannot be held responsible for damage caused by improper or careless use. The unit is not intended for commercial use, nor for use in medical clinics etc.
- The unit should not be used for longer than 30 – 40 minutes without interruption.
- Never operate the unit unattended.
- Never stand in the foot bath.
- Always switch the foot bath off and remove the plug from the mains socket before moving it or cleaning it. Always make sure your hands are dry.

- Children are not aware of the dangers associated with electrical appliances. Make sure that the foot bath cannot be used by children without supervision.
- Never immerse the device in water.
- To fill the foot bath, it must not be placed in the shower, bath tub or wash basin.
- The device must not be connected to the power supply while being filled.
- Consult your doctor if you have any medical condition which affects the legs or feet, e.g. phlebitis, varicose veins or diabetes.
- The water must be drained only through the drain holes.
- The surface of the appliance is hot, persons who are insensitive to heat must use it with particular care.
- Do not operate the foot massager without water.

Disposal

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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