## DEUTSCH

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## ENGLISH

**Important safety information – retain for later use.**

In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.
- Children should not use the thermometer without supervision.
- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.
- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- Using the thermometer in strong electromagnetic fields such as next to a mobile phone, can cause a malfunction.
- When the thermometer is switched on, it carries out a self-test. Therefore the accuracy of measurement need not be tested at regular intervals.
- If the temperature is very abnormal, please contact your family doctor promptly.
- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. For more details, please contact customer service at the address indicated.

**Measuring methods:**

**Temperature measurement in the anus (rectal)**

This method of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

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